


# Motif Aura Glow™ Quick Guide

## 1. Buttons & Features


- A Nightlight Power Button
- B Level Down (suction)
- C Play/ Pause Button
- D Level Up (suction)
- E Mode Button
- F Level Display
- G Power Display
- H Time/ Minute
- I Time Display

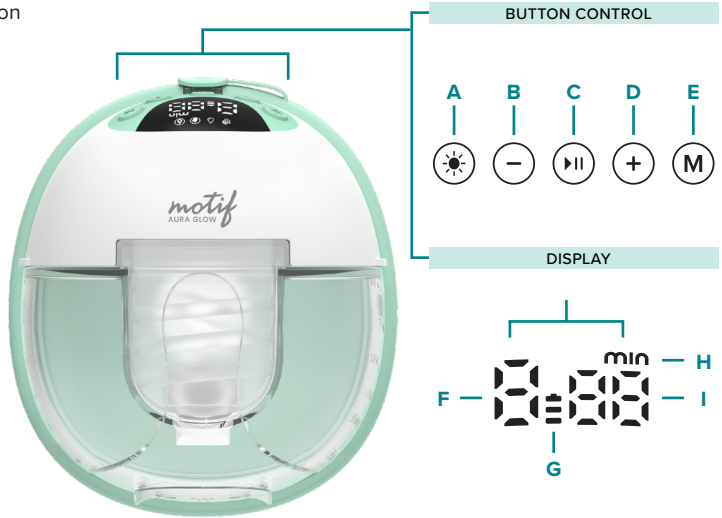

## 2. Assembling the Motif Aura Glow™ Pump

### Assembling Pump

1. Wash hands before touching breast pump components.
2. Install the valve and diaphragm in the collection container.
3. Attach the breast shield to the collection container, pressing the edges to seal.
4. Push a flange insert into the breast shield if using.
5. Attach the pump motor to the collection container.

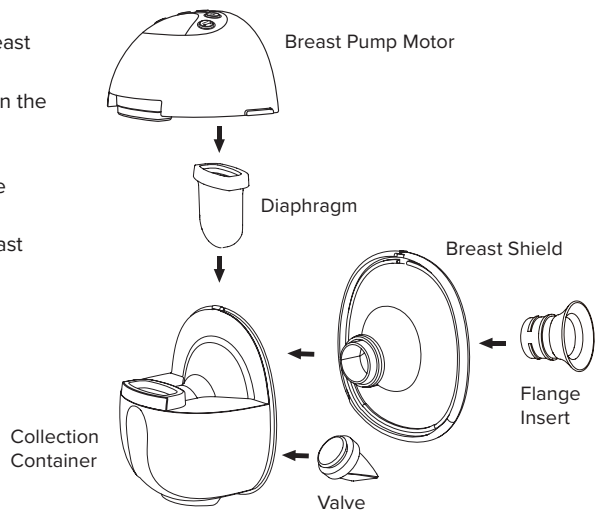


### 3. Expressing Your Milk

1. Long press the PLAY/PAUSE button to start the breast pump. The screen will light up.
2. Short press the PLAY/PAUSE button to start the pump. Press the MODE button to switch modes after starting the pump.
  - Massage, Expression and General Pumping Mode: 1~9 levels
  - Cluster Feeding Mode: 1~5 levels
3. Short press the nightlight power button key to turn the nightlight on to check milk levels.
4. Long press the PLAY/PAUSE button to turn off the breast pump.

### 4. After Pumping

1. When you are finished expressing, long press the PLAY/PAUSE button to turn off the pump and slide a finger between your breast and the breast shield to separate them. Carefully remove the pump from your breast keeping it upright.
2. Gently pull back on the top of the breast shield and pull the pump motor off the collection container using the wings on the side.
3. Remove the stopper from the pour spout. To avoid spilled milk, carefully align the pour spout with the bottle and pour milk into the bottle.

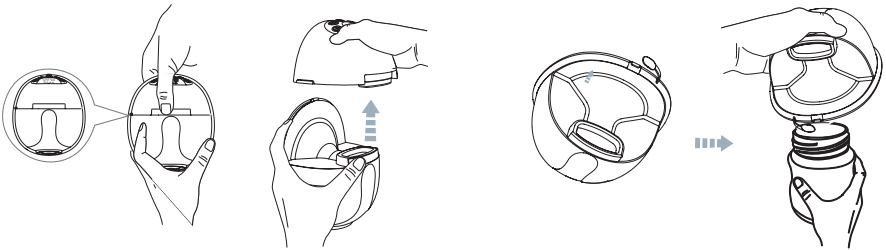

### 5. Warnings & Precautions



**WARNING:** Potentially dangerous situation that could result in serious injury.



**CAUTION:** A possibly dangerous situation that could result in slight bodily injury.



Manufactured for:  
**Motif Medical®**, LLC  
8 The Green, Suite 5601  
Dover, DE 19901

Phone: (844) 272-8390  
Email: [info@motifmedical.com](mailto:info@motifmedical.com)  
Web: [www.motifmedical.com](http://www.motifmedical.com)

*motif*  
AURA GLOW™